IT - Indurent Gel

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EN - Indurent Gel

POLYSILOXANE-SUPPORTED CATALYSTS (C-SILICONES) USE

Indurent Gel is used together with the following Zhermack products: Zetaplus, Zetaplus Soft, Oranwash VL, Oranwash L, Thixoflex M, Orthogum, Zetalabor and Titanium. For applications refer to the dedicated instructions that are found in the relative packaging.

WARNINGS / PRECAUTIONS

- Tightly close Indurent Gel after use.

- Wear gloves and protective clothing. Avoid contacting the skin and eyes with the catalyst; in case of accidental contact with skin, wash thoroughly with soap and water. In case of contact with eyes, rinse thoroughly with running water and consult a doctor. Consult a doctor immediately if accidentally swallowed.

- Avoid staining lab coats or clothing. Take off clothing in the case of contamination.

- Not recommended for use on pregnant women.

- Stop using Indurent Gel if irritation, redness or other signs of hypersensiti- vity occurs during use.

STORAGE AND GUARANTEE

The material is guaranteed for a period of 36 months if stored correctly between 5° - 27° C (41° - 80° F).

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