R-080093 / R-080095

MULHACEN

E.....	RECORTADORA DE MODELOS CON FRENO.....	5
GB ...	MODEL TRIMMER WITH BRAKE.....	8
F.....	TAILLE-PLATRE AVEC SYSTEME DE FREIN.....	11
D.....	MODELL TRIMMER MIT BREMSE.....	14

MESTRA®

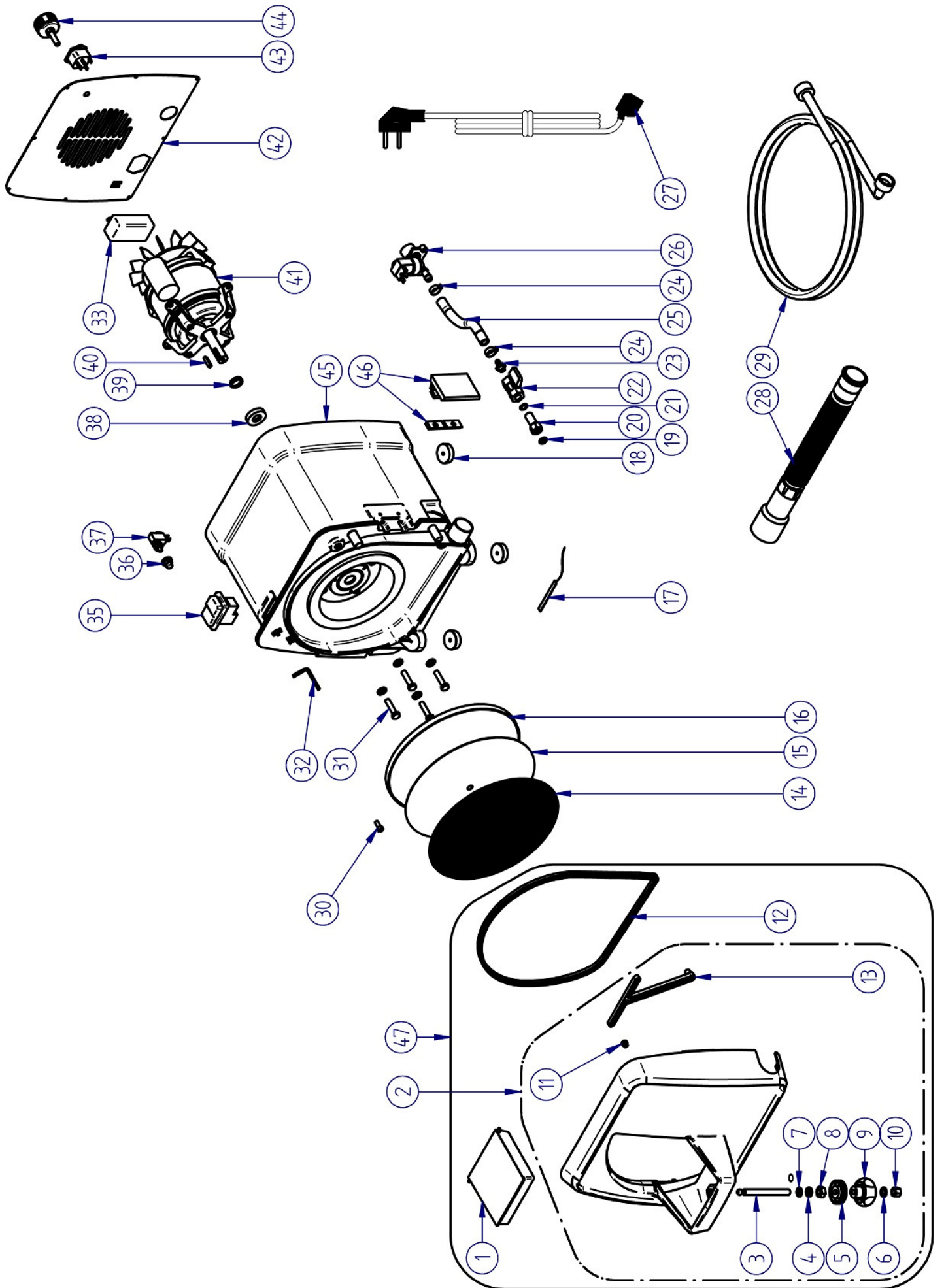
TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Nº	Ref.	Descripción	Description
1	080093-35	Mesa basculante	Tilting table
2	080093-74	Tapa delantera con aspersor y elevación	Front cover with sprinkler and elevation
3	080093-36	Espárrago mesa elevación	Elevation table stud
4	080093-23	Casquillo plástico espárrago	Stud plastic bushing
5	080093-02	Tuerca regulación altura	Height regulation nut
6	080093-58	Arandela bloqueo	Lock washer
7	080093-24	Junta goma mesa	Table rubber gasket
8	080093-41	Tuerca de plástico	Plastic nut
9	080093-06	Pomo mesa elevación	Elevation table knob
10	080093-17	Tuerca pomo elevación	Elevation knob nut
11	000110	Goma tope microinterruptor	Microswitch top rubber
12	080093-10	Perfil goma cierre puerta	Cover rubber strip
13	080093-37	Aspersor	Sprinkler
14	080093-04	Disco de lija grano 80 grueso (5 U.)	Sandpaper disc grain 80 gross (5 U.)
	080093-05	Disco de lija grano 120 fino (5 U.)	Sandpaper disc grain 120 fine (5 U.)
	080093-55	Disco de diamante (1 U.)	Diamond disc (1 U.)
15	080093-03	Disco velcro adhesivo	Adhesive velcro disc
16	080093-39	Plato aluminio para Velcro	Aluminium plate for Velcro
17	080093-11	Tira de LED	LED strip
18	080080-26	Pata	Leg
19	080093-20	Junta tórica aspersor 8,5x14,5	Sprinkler o-ring 8,5x14,5
20	080093-18	Tornillo aspersor	Sprinkler bolt
21	080093-19	Junta tórica tornillo aspersor 10x14	Sprinkler bolt o-ring 10x14
22	080093-13	Grifo caudal agua	Water flow valve
23	080093-14	Espiga para grifo	Dowel for water valve
24	000304	Abrazadera	Clamp
25	080093-42	Tubo reforzado	Reinforced pipe
26	080080-23	Electroválvula 230 V	Electrovalve 230 V
	080080-23B	Electroválvula 115 V	Electrovalve 115 V
27	000250	Cable 230 V	Cable 230 V
	000252	Cable 115 V	Cable 115 V
28	080087-07	Manguera desagüe	Drain hose
29	080080-01	Manguera entrada agua	Water hose
30	030120-03	Tornillo sujeción plato	Plate holding bolt
31	080093-48	Tornillo sujeción motor	Motor holding bolt
32	080410-63	Llave Allen	Allen wrench
33	080093-53	Fuente de alimentación 12 V	Power supply 12 V
35	080080-40	Interruptor de seguridad 230 V	Safe switch 230 V
	080080-40B	Interruptor de seguridad 115 V	Safe switch 115 V
36	080093-22	Capuchón microinterruptor	Microswitch hat
37	080093-16	Microinterruptor final de carrera	End stroke microswitch
38	080093-09	Retén eje motor	Motor shaft seal
39	080093-46	V-ring	V-ring
40	080093-25	Chaveta	Feather key
41	080093-01	Motor 230 V	Motor 230 V
	080093-01B	Motor 115 V	Motor 115 V
42	080093-71	Tapa trasera	Back lid
43	080422-18	Base de conexión con filtro de red	Connection base with net filter
44	080093-43	Extractor del plato	Plate extractor
45	080093-33	Carcasa trasera	Rear housing
46	080093-21	Pestillo	Latch
47	080093-73	Tapa delantera completa	Complete front cover

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The R-080093/R-080095 is an easy to handle model trimmer specially designed for the field of dental prosthesis.

Its 3000 r.p.m. powerful motor has a brake that stops the spin after turning the main switch off.

Its dimensions give the unit an excellent stability during operation. It features a 122 mm window (one of the widest in the market).

Its design allows to place a big storage tray (R-100062) on the unit in order to keep the models well arranged and the workbench clear.

The R-080095 has a factory installed diamond disc.

The R-080093 has a factory installed 80 grit abrasive disc.

With the R-080093, besides you will receive two additional sample discs (grain 80 and 120)

Replacements:

80 grit abrasive disc (coarse)	R-080093-04 (5 u.)
120 grit abrasive disc (fine)	R-080093-05 (5 u.)
Diamond disc	R-080093-55 (1 u.)

The following items are also factory supplied:

- A water inlet hose.
- An extensible drainpipe.
- An Allen spanner to dismount the plate.
- A bolt to dismount the plate.

INSTALLATION OF THE UNIT

1. Carefully unpack the model trimmer. Make sure the unit has not suffered any damage during the transport.
2. Place the unit on a flat and firm surface, away from any heat or vibration source. The place

has to be close to a water source with a threaded connection and to a drainpipe.

3. Connect the bent end of the flexible hose to the fitting placed at the rear of the unit. Connect the other end to the water inlet. To do this, insert the threaded tap connector at the end of the hose into the tap.
4. Connect the water outlet hose into the water outlet (H) in the right bottom front part of the unit. Connect the other end of the hose to a drainpipe. Place the hose with a certain slope, to prevent the water flow to stop and to ease the drainage.
5. For an optimum operation of the unit Talleres Mestraitúa, S.L. strongly recommends to use the R-080085 plaster decanter.
6. The unit has four rubber pads. When designing the unit, special care has been taken to give the unit enough weight and robustness to prevent any movement during its operation. No additional fixing element is needed.
7. Connect the unit to a proper grounded power source.
8. Make sure that the disc cleaning system works properly. To do this, turn on the main switch in the upper part of the unit. The trimmer will start up and the electro-valve that monitors the inlet water flow will open.
9. Then open the water stopcock and select the desired flow by means of the tap knob of the trimmer. After a few minutes of operation, make sure there is no leak in the pipe and hose connections.

OPERATION OF THE UNIT

1. Adjust the table (B) with the knob (E), according to the type of work. Once the working position has been selected, screw the locknut (C) up as far as it goes to fix the table.
2. If you want to trim at a vertical 90° angle, align the lower edge of the table with the edge of the casing:

3. Take proper safety measures. Always wear:
 - Use safety goggles.
 - Use safety gloves.
 - Use a face-mask
 - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
 - Be careful with long hair.
4. Turn on the main switch placed in the upper part of the unit. The motor will start up and the operation light (E) will lit up.
5. Open the water stopcock. Then use the tap knob of the trimmer (G) to obtain the desired water flow.
6. Place the model to be trimmed on the table (B). For a greater precision while trimming, the table has a set of marks to be used as reference to determinate the cutting angles.
7. When the work is over, turn the main switch off.
8. After operating the trimmer, it is strongly recommended to clean the table and the front casing with a damp cloth to remove the remains of plaster.

ABRASIVE DISC REPLACEMENT

When the abrasive element tears or wears out it must be replaced. **MESTRA** spare-parts should be always used, since they are the only ones that assure a perfect balance of the unit and top quality. To replace the disc, proceed as follows:

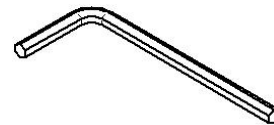
1. Open the lock (F) and the front cover. The safety micro-switch will prevent an eventual start-up of the motor.
2. Remove the worn out disc and place a new one. Make sure it is properly centered.
3. Close the cover again, making sure the lock (F) is properly closed.


ALUMINIUM PLATE MAINTENANCE

If plaster accumulates in the back of the aluminum plate, then the machine could vibrate. Then proceed to clean as follows:

1. Open the lock (F) and the front cover. The safety micro-switch will prevent an eventual start-up of the motor. For extra safety, disconnect the unit from the power source and close the water stopcock.
2. Remove the abrasive disc.
3. Use the Allen spanner to unscrew the bolt that fixes the disc to the motor shaft.

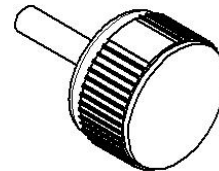4. Remove the extraction bolt from the back of the unit and screw it up in the middle of the plate. Then pull the screw and extract the plate.

5. Dip the plate in water, remove the plaster with a brush and dry.
6. When you place the plate again, do not forget the pin that couples it to the motor.

MAINTENANCE AND CLEANING

To ensure a correct, reliable and durable performance of the unit we strongly recommend:

- After using the unit, clean the dust and plaster remains from the table (B) and the front cover.
- Do not use alcohol or any other solvent to clean the unit.
- Never immerse the unit in a liquid.
- If you notice unusual noises or vibrations, send the unit to a technical assistance service.
- Replace the abrasive disc whenever you notice a significant fall in performance.
- After replacing the disc, check that there are not unusual vibrations.
- The unit has been designed to operate correctly leaning on the four rubber blocks in its base. These elements are strong enough to bear the movement of the trimming.

PRECAUTIONS

- ◆ Before connecting the unit to the power source, make sure the voltage is correct.
- ◆ When trimming a model, protect yourself suitably. Always wear gloves, safety goggles, and a mask. Use proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine. Be careful with long hair.
- ◆ Always use the water-cleaning system to clean the discs.
- ◆ Do not allow children or unskilled personnel to operate the unit.
- ◆ Do not allow anybody to by-pass the safety micro-switch that detects that the cover/door is closed or to operate the unit when the cover/door is open.
- ◆ If the motor shaft jams during normal operation, disconnect the unit from the power source before trying to free the shaft by forcing its rotation manually.
- ◆ Do not polish, brush, or grind materials that could release particles that are toxic when inhaled.
- ◆ Before using the unit, make sure it is placed on a firm and solid base.

- ◆ Do not use the unit close to solvents or other inflammable materials.
- ◆ When replacing the disc, make sure that the sprinkler (I) is in good condition and free from plaster.
- ◆ When installing the unit, make sure that the disc-cleaning circuit is perfectly leak-proof.
- ◆ Do not trim materials for which the unit has not been designed, especially metal objects.
- ◆ Do not handle or adjust the table when the unit is working.

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GUARANTEE

All **MESTRA**[®] units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA**[®] will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

MESTRA[®] will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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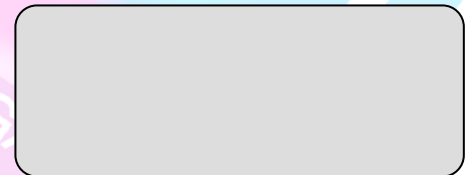
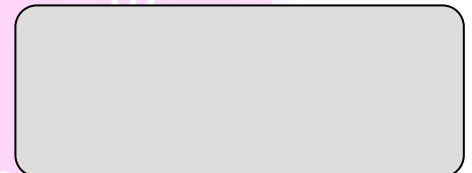
TALLERES MESTRAITUA S.L. is a company manufacturing tools and appliances for the dental prosthesis sector, from the business address shown at the bottom of the page.

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I HEREBY STATE: That the MODEL TRIMMER WITH BRAKE R-080093 / R-080095, with the following characteristics:

Potencia / Power:	1600 W / 2 HP
Velocidad / Speed:	3000 rpm
Tensión / Voltage:	230 V, 50/60 Hz 115 V, 50/60 Hz
Ø abrasivo / Abrasive track Ø:	250 mm
Alto / Height:	340 mm
Ancho / Width:	420 mm
Largo / Length:	345 mm
Peso / Weight:	14.5 kg

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Meet the low voltage requirements set out in Directive 2014/35/UE, in Directive 2014/30/UE concerning electromagnetic compatibility and in Directive 2006/42/EC concerning machinery safety.

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In witness whereof, the company issues this declaration of conformity in Sondika, on April 21st 2016.

Firma / Signed:

Ignacio Mestraitua,
Director Gerente / Managing Director

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA