R-080132 / R-080134 / R-080138

E.....	HORNOS DE PRECALENTAMIENTO - "CONCEPT"	6
GB.....	PREHEATING FURNACES - "CONCEPT"	10
F.....	FOURS DE CHAUFFE - "CONCEPT"	14

MESTRA®

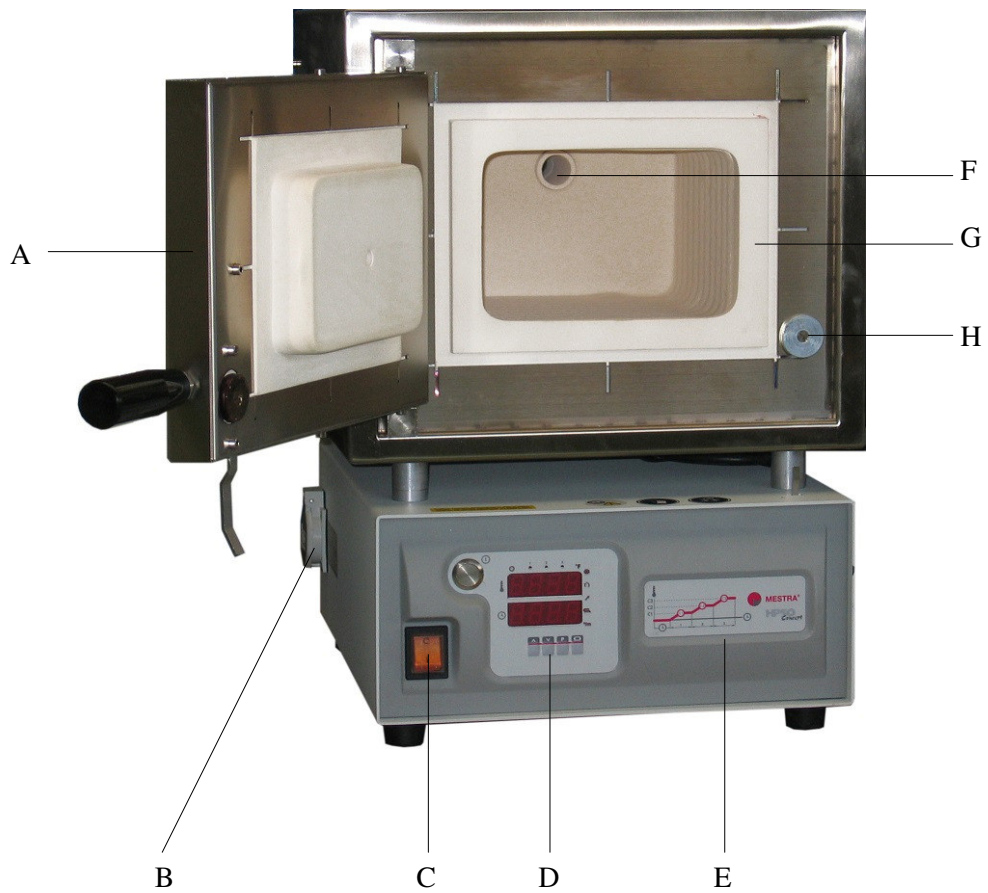
TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

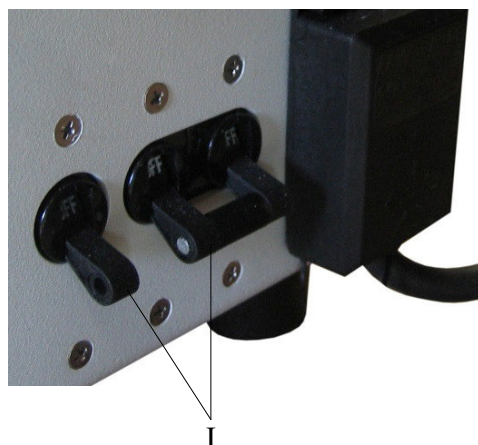
Tel. (+34) 944530388 - Fax (+34) 944711725

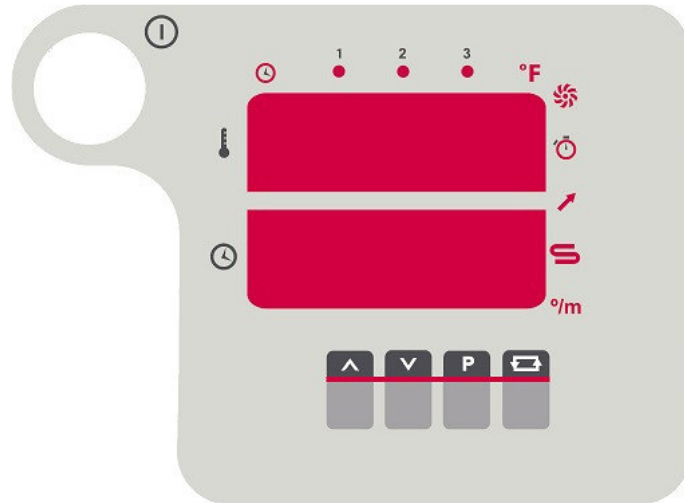
mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

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R-080132, R-080134 and R-080138 are three practical furnaces especially designed to heat casting rings and crucibles for the dental technicians. They have a control panel on which a pre-set sequence of temperatures and times can be programmed by the user. Furthermore, up to 4 different programmes can be stored. This simplifies programming, as previous programmes can be recovered when required

INSTALLATION

- Carefully unpack the unit. Inside the box there should be an SiO₂ hearth plate. Check also that the muffle lining the inside of the furnace has suffered no damage during transport.
- Set the furnace up on a smooth, firm, horizontal base away from any source of vibration. Allow enough space for ventilation.
- There should be no other objects less than 15 cm from the unit. Ensure especially that the ventilation tube is free and at least 20 cm from the closest object or wall.
- During normal furnace operation fumes may filter through the ventilation tube. This tube should therefore be placed below an extractor hood.
- Connect the unit to a 230 V, 50/60 Hz, earthed power outlet which protected by a thermal magnetic cut-out with a suitable amp rating (consult your installer).
- Before using the furnace heat it up to 300 °C and hold it there for one hour to evaporate off any moisture in the muffle.

SMOKE EXTRACTOR

At the left-hand side of the machine there is a 230 V current socket, extremely useful for connecting up a suction system. We recommend using the Mestra extractors R-080124 and 080125, specifically designed for this function. By default, the extractor's control output operates when temperature is between 75 °C and 650 °C. In order to modify these limits, proceed as follows:

- Briefly press key . Then, keep pressing it for some seconds.
- Icon will blink.
- Upper display with show "HP".
- Using keys **▲▼**, select the temperatura you want the extractor stops (by default, 650 °C).
- Briefly press key .
- Upper display with show "LO".
- Using keys **▲▼**, select the temperatura you want the extractor starts (by default, 75 °C).
- If you set "- - - -" the extractor will never work.
- Briefly press key to confirm. Then, keep pressing it for some seconds to finish configuration.

ALARM

The furnace allows to activate an acoustic alarm to warn when the program is finished. To activate or deactivate the alarm, proceed as follows:

- Briefly press the key . Then keep pressing it for some seconds. Pilot will blink.
- Briefly press the key . Pilot will blink.
- Use the keys the **▲▼** to select "YES" or "NO" on the upper display in order to activate or deactivate the alarm.
- Briefly press the key to shift to the lower display.
- Select the type of alarm (1 or 2) by pressing the **▲▼** keys.
 - The alarm will go on only once.
 - The alarm will go on every 2 minutes.
- Briefly press key to confirm. Then, keep pressing it for some seconds to finish configuration.

MODE ^o/m

The furnace allows to program the heating times in minutes or in grades per minute. In order to choose one mode or the other, proceed as follows:

- Briefly press the key . Then keep pressing it for some seconds. Pilot will blink.
- Briefly press the key . Pilot will blink.
- Briefly press the key . Pilot **%/m** will blink.
- Use the keys **▲▼** to select, on the upper display: "YES" to program in grades/minute. "NO" to program in minutes.
- Briefly press key to confirm. Then, keep pressing it for some seconds to finish configuration.

DELAYED START

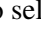
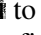
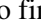
The furnace allows to program a delayed start. In order to choose a delayed start proceed as follows:

- Briefly press the key . Then keep pressing it for some seconds. Pilot will blink.
- Briefly press the key . Pilot will blink.
- Briefly press the key . Pilot **%/m** will blink.
- Briefly press the key . Pilot will blink.
- Use the keys **▲▼** to select, on the upper display: "YES" to program a delayed start. "NO" to erase delay.
- Briefly press key to confirm.
- Use the keys **▲▼** to select, on the lower display, the number of hours you want the furnace keeps waiting before start.

- Briefly press key  to confirm. Then, keep pressing it for some seconds to finish configuration.
- Then, program the furnace as indicated in section “Programming”.

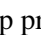
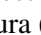
MODE °C/°F

The furnace allows to see the temperature in grades Celsius or Fahrenheit. In order to choose one mode or the other, proceed as follows:

- Briefly press the key . Then keep pressing it for some seconds. Pilot  will blink.
- Briefly press the key . Pilot  will blink.
- Briefly press the key . Pilot  will blink.
- Briefly press the key . Pilot  will blink.
- Briefly press the key . Pilot  will blink.
- Use the keys  to select, on the upper display: “YES” for °F.
“NO” for °C.
- Briefly press key  to confirm. Then, keep pressing it for some seconds to finish configuration.

PROGRAMMING

In order to program a heating sequence into the furnace hearth and then save it as one of the 4 memories which can be stored, proceed as follows:

- Briefly press key .
- Simultaneously keep pressing keys . Pilot for step **1** will light.
- Use keys  to select the time (minutes) to reach the first set temperatura (C1). If you are working in mode °/m, select the heating speed (from 1 to 15 °C/min).
- Briefly press key  to confirm.
- Use keys  to select the set temperatura C1 that you want to reach in step 1.
- Briefly press key  to confirm.
- Use keys  to select the time (minutes) you want to keep temperature C1.
- Briefly press key  to confirm. Pilot for step **2** will light.
- Proceed as previously to program steps **2** and **3**.

If, by any reason, you want to omit one of the steps, you must simply select the value “- -” for the time. This means that program will finish in this step.

STORING PROGRAMS IN THE MEMORY

The electronic controls of the Concept furnaces allow you to store up to 5 different dosages for each type of material. To store a program, proceed as follows:

- First of all, select the program you want to store, as explained in previous sections.

- Once you have selected the values you want to store, select the program (*P1*, *P2*, *P3*, *P4* or *Pr*) in which you want to store the dosage by pressing the **P** key as many times as required.
- Finally, keep the **P** key pressed for a few seconds. The display will flicker once to indicate that the values have been stored in the memory.

Memory *Pr* is reserved for the direct heating program (look at the next section).

If you want to save a program with delayed start, follow previous sections in this order:

- a) **Delayed start.**
- b) **Programming.**
- c) **Storing programs in memory.**

DIRECT HEATING PROGRAM

A direct heating program has only one heating time, only one set temperature and only one keeping time. Everytime you turn the furnace on, the direct heating program will be loaded by default. This situation is indicated by pilot .

In order to modify the values of this direct heating program, simply program a cycle with only one rise and save it in memory *Pr* (look at previous section). These new values will be loaded everytime you turn the furnace on. If, by error, you try to save a program with several rises in memory *Pr*, nothing will be saved.

USING THE PROGRAMS STORED IN THE MEMORY

- To use a program stored in the memory, press the **P** key as many times as required to select the program you want (*P1*, *P2*, *P3*, *P4* or *Pr*).
- Press the key  to display the values stored.
- Press the **START/STOP**  key to start.

If you want to use a stored program but including a delayed start, proceed as following:

- a) Press the **P** key as many times as required to select the program you want (*P1*, *P2*, *P3*, *P4* or *Pr*).
- b) Press the **START/STOP**  key to start.
- c) Keep pressing the **START/STOP**  key to stop the furnace. This way, the program data will keep in memory.
- d) Now, follow steps of the section “Delayed start”.
- e) Press the **START/STOP**  key to start.

OPERATION

- Place the SiO₂ plate on the hearth, taking care not to damage the muffle.

- Action the main switch on. Some of the controls on the panel should flash to indicate that the furnace is ready to operate.
- Put in the cylinders and crucibles to be heated. When the furnace is operating, always use gauntlets and tongs. Talleres Mestraitua, S.L. recommends MESTRA tongs.
- The furnace is factory-supplied with a series of standard programs in its memory. However, it is advisable to program the furnace as indicated above to ensure that the programs stored meet your own requirements. These programs can be used and modified as required in future.
- Select the program you want to run (*P1, P2, P3, P4 o Pr*). If you do not select any program, then the direct heating program will run (look at the previous section).
- Press the **START/STOP** Ⓞ button to start the program.
- In order to stop the running program at any time, keep pressing the **START/STOP** Ⓞ button.
- If there is a power cut the furnace retains in its memory the parameters of the step in progress, and when power is restored the program will continue from the point at which it stopped. If the switch is turned off before the program is completed, the furnace takes this as a power cut.
- If alarm is activated, it will sound at the end of the program (look at the *Alarm* section).
- The furnace will keep the last set temperature until the user stops the program running by keep pressing the **START/STOP** Ⓞ button.
- At the end of the process, remove the cylinders and crucibles carefully. While the unit is running the outer housing may heat up. The cylinders and crucibles also heat up considerably, and so gauntlets and tongs should be used when handling them.
- To run another program, first you must stop the furnace by keep pressing the **START/STOP** Ⓞ button.
- Turn the main switch off to disconnect the furnace from the power supply.

DISPLAY MESSAGES

The Concept furnaces can show some messages in display, to make the use easier:

- dEL** It means that the delayed start time is counting down. Lower display shows the remaining hours for the start.
- doo** It means that the door is open.
- tH1** It means that the thermocouple is damaged or unplugged. Replace the thermocouple.

- tri** It means that the triac is damaged. Turn the main switch off and contact immediately to technical service.

MAINTENANCE AND CLEANING

To ensure long, reliable operation of the unit, proceed as follows:

- Clean the outside of the furnace regularly with a cloth moistened in soapy water.
- Take care not to damage the muffle lining the inside walls of the furnace hearth when you put in or take out cylinders and crucibles.
- Always use the SiO₂ hearth plate to prevent filtration, which may damage the muffle.
- Prevent liquids or solids from contaminating or dirtying the muffle of the furnace.
- To make best use of the unit, Talleres Mestraitua, S.L. recommends the use of MESTRA cylinders and crucibles.
- The two circuit breakers located near the cable protect the furnace against electric overloads. To get the furnace working again, once solved the problem, you will have to reset the circuit breakers.
- If the controls detect that the triac is damaged, upper display will show the message “tri”. If this alarm message appears, turn the main switch off and contact immediately the technical service.

IMPORTANT

If during the normal use of the furnace the upper display shows the message “tH1”, turn the main switch off and ask immediately to an authorized MESTRA distributor. This message means that the thermocouple is damaged. The thermocouple is subject to wear, and its durability depends on working temperature. This element is not covered by guarantee.

PRECAUTIONS

- Before connecting the unit ensure that the power outlet is for 230 V, 50/60 Hz, and that it is earthed. It must also be properly protected by a thermal magnetic cut-out (consult your installer).
- Do not allow children or unskilled personnel to handle the unit.
- Always keep the door closed while the furnace is operating.
- Take care when opening the door: the hot fumes pouring out can cause burns.
- Do not overfill the furnace or place objects in it which might fall when the door is opened.
- Do not put inflammable materials or liquids in the furnace, or substances which may give off toxic fumes.

- Place the furnace on a flat, firm, vibration-free surface close to an extractor hood. Ensure that the furnace is well away from any flammable materials.
- Place the unit so that its sides are at least 15 cm from any wall or other object. Take particular care to place the ventilation tube in a free area at least 20 cm from the closest object.
- Always use gloves and tongs to remove and put in cylinders and crucibles. Do not touch the metal housing of the furnace when it may be hot.
- Handle heated materials removed from the furnace with great care: their temperature may be as high as 1100 °C and may therefore cause accidents.
- Before using the furnace for the first time, hold it at 300 °C for 1 hour to eliminate any moisture in the muffle.
- Always use the furnace under an extractor hood.

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