R-080235

E	ARENADORA AUTOMÁTICA GALAXY	3
GB ...	GALAXY AUTOMATIC SANDBLASTER.....	7
F	SABLEUSE AUTOMATIQUE GALAXY	11
D	GALAXY SANDSTRAHLAUTOMAT	15
P	AREADORA AUTOMÁTICA GALAXY	19
I	SABBIATRICE AUTOMATICA GALAXY.....	23

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. + 34 944530388 - Fax + 34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

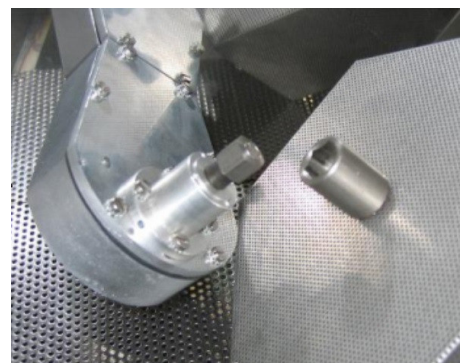
Fig. 1

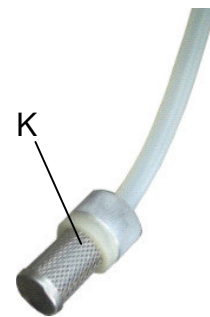
Fig. 2

A	Pistola sopladora / Blasting gun / <i>Buse et porte-buse</i> / Injektor / Pistola soffiatrice / <i>Pistola assopradora</i>
B	Cesta / Basket / <i>Panier</i> / Korb / Cestello / <i>Cesto</i>
C	Manguitos / Sleeves / <i>Manchons</i> / Durchgriffe mit Schläuchen / Manicotti / <i>Mangas</i>
D	Interruptor luz / Light switch / <i>Interrupteur lumière</i> / Lichtschalter / Interruttore luce / <i>Interruptor luz</i>
E	Entrada de aire / Air intake / <i>Entrée d'air</i> / Lufteintritt / Ingresso aria / <i>Entrada de ar</i>
F	Toma de corriente / Power outlet / <i>Prise de courant</i> / Stromanschluss / Presa corrente / <i>Tomada de corrente</i>
G	Manorregulador / Pressure regulation valve / <i>Mano régulateur</i> / Druckregler / Regolatore pressione / <i>Regulador de pressão</i>
H	Manómetro / Pressure gauge / <i>Manomètre</i> / Manometer / Manometro / <i>Manómetro</i>
I	Temporizador / Timer / <i>Temporisateur</i> / Zeitschalter / Timer / <i>Temporizador</i>
J	Botón funcionamiento manual / Manual operation button / <i>Bouton mode manuel</i> / Handbetriebstaste / Pulsante funzionamento manuale / <i>Botão funcionamento manual</i>
K	Filtro / Filter / <i>Filtre</i> / Filter / Filtro / <i>Filtro</i>

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The Galaxy R-080235 is an automatic sand blaster specially designed for sandblasting small pieces within the field of dental prosthesis on continuous automatic mode or ion manual mode. In order to achieve an optimal operation and performance of the unit, we strongly recommend you to read carefully the following instructions:

INSTALLATION

- Carefully unpack the unit. You will find the following accessories inside the cabin:
 - A Ø 4 / 6 mm pipe to connect the unit to the air supply.
 - A 90° PVC elbow to be fitted into the outlet aspiration hose.
 - Two plastic covers to protect the glass (R-080235-50).
 - A basket.
- Choose a firm, flat surface to place the unit, far from heat or vibration sources. The room atmosphere should never be damp. An excess of humidity could compact the aluminium oxide particles inside the cabin and, therefore, obstruct the blasting nozzles. Finally, place the unit at a height that allows an easy manipulation of the objects inside the cabin.
- Fit the connecting pipe supplied with the unit into the inlet adaptor (E) placed at the right side panel of the machine. The other end should be fitted into an intake of pressured air fed by a compressor with a minimum pressure of 7 bar (0.7 MPa) and a minimum flow of 100 litre/minute.
- Connect the power cord to a grounded 230 V, 50/60 Hz power source.
- Fill the hopper in the cabin with aluminium oxide of a maximum diameter of 300 micron. To obtain a maximum autonomy, load the hopper up to a maximum height of 2 cm below the tray.

NOTE: the unit is optimized for 200 micron particles and getting an stable jet. With smaller particle diameters (100 microns) the jet may be more irregular.

- **The unit is factory supplied with a 3 mm boron carbide nozzle (R-080235-25), material much more abrasion-resistant than the traditional tungsten.**

PRESSURE REGULATION

The blowing pressure of the blasting gun can be regulated at will between 2 and 5 bar (0.2 and 0.5 MPa), according to the nature and special features

of the work to be done. Take into account that the highest pressure will be limited by the maximum pressure of the air supply we are using. In any case, the pressure should never be higher than 6 bar (0.6 MPa).

The unit features a pressure regulation valve (G) to select the working pressure easily. To adjust the pressure pull the valve outwards and turn it until the intended pressure has been reached. The value of the pressure is shown in the built-in pressure gauge (H). After adjusting the pressure, block the valve knob by pressing it inwards.

The pressure gauge shows the gun's blowing pressure at any time, even when the gun is not being used.

CONNECTING TO THE ASPIRATION UNIT

The blasting cabin should always be used connected to a pressured aspiration unit. For this purpose there is an outlet hose on the upper left side panel of the unit. The aspiration unit to be used has to create a light depression inside the cabin for the dust particles to circulate smoothly and continuously towards the outlet hose.

NOTE: make sure that the aspiration unit used can suck abrasive particles. We strongly recommend the MESTRA Eolo Plus R-080531 aspiration unit.

The factory-supplied PVC elbow can be adapted according to the needs and shape of the pressured aspiration unit to be used.

OPERATION

The Galaxy R-080235 automatic sandblaster can be operated on automatic or manual mode.

Automatic mode:

1. Fit the basket (B) into the motor shaft (Fig. 1).

Never force the motor shaft to rotate by hand, whether the basket is fitted or not. The motor's inner mechanism could be damaged.

2. Put the pieces you want to sandblast inside the basket.
3. Position the gun (A) facing the pieces, bearing in mind that the basket will rotate clockwise. At the beginning the motor may start to rotate anti-clockwise, but before completing its first turn it will start to rotate clockwise.

4. Direct the sandblast accurately to prevent the sand going straight to the cabin walls. The sandblast could damage the cabin.
5. Close the cabin lid making sure the glass fits tightly into its frame.
6. Turn on the pressured aspiration unit.
7. For a better vision of the piece, turn on the light inside the cabin by pressing the switch (D).
8. To select the automatic operation time, turn the timer (I) clockwise (maximum 30 minutes).
9. The basket will start to rotate and the gun will blast aluminium oxide automatically.
10. If you want to stop the automatic operation, turn the timer (I) anti-clockwise.

Manual mode:

The blasting gun can be operated without rotating the basket or using the timer.



1. The basket can be removed for an easier operation (on manual mode the motor does not rotate).
2. Open the cabin lid and put the piece you want to manipulate inside it. The pieces can be introduced through the sleeve hole (C) as well. Make sure the level of the aluminium oxide in the hopper is correct.
3. Close the cabin lid making sure the glass fits tightly into its frame.
4. Protect yourself properly: use protection goggles, gloves and mask.
5. Turn on the pressured aspiration unit.
6. For a better vision of the piece, turn on the light inside the cabin by pressing the switch (D).
7. Position the piece to face the blast by manipulating it through the built-in sleeves (C) inside the cabin.
8. For the gun to start blasting aluminium oxide press the button (J). On manual mode the motor does not rotate.

MAINTENANCE AND CLEANING

In order to achieve a long working life of the machine, we strongly recommend you:

- Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide.
- Preferably use dry-piston air compressors. The presence of oil particles in the air could make the sand compact and obstruct the gun's nozzle.
- To prevent the inner walls of the cabin wearing away due to abrasion, do not aim the aluminium oxide blast straight at the cabin walls.
- Periodically remove the aluminium oxide from the cabin. To do this, use the drain plug in the bottom of the hopper.
- For an overall cleaning of the unit, use a dry cloth or moistened with a proper product.

- In case of obstruction of the gun, dismount the nozzle holder. Clear the nozzle by blowing in the opposite direction to the air flow, or, if needed, by using a pin or a thin wire. Then set the gun again, making sure it is correctly centred.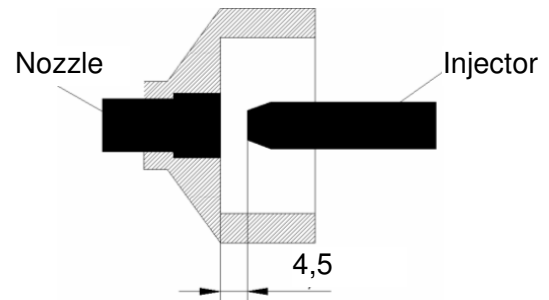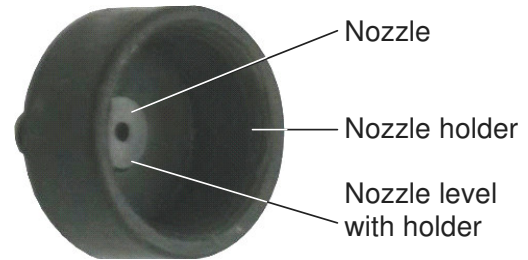

If the nozzle came loose, glue it to its holder with cianocrilate. The nozzle should be level with the inner surface of the nozzle holder, to make sure that the distance from the injector to the nozzle is 4.5 mm and the Venturi effect takes place.

- Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide. The gun (Ø 3 mm nozzle) can blast aluminium oxide up to particles of Ø 300 micron. If you use aluminium oxide particles bigger than Ø 300 micron, you may need to remove the filter (K) placed at the end of the gun's pipe, inside the hopper (Fig. 2). Without any filter obstructions in the nozzle will be more frequent.

PRECAUTIONS


- ◆ Never exceed a 6 bar pressure (0.6 MPa). The optimal working pressure lies between 2 and 5 bar (0.2 and 0.5 MPa).
- ◆ Never force the motor shaft to rotate by hand, whether the basket is fitted or not. The motor's inner mechanism could be damaged.
- ◆ Always work with the cabin lid closed and correctly protected. Use protection mask and goggles.
- ◆ Connect the cabin's outlet hose to an aspiration unit.
- ◆ If the protection sleeves (C) tear or wear, replace them immediately..
- ◆ Do not allow children or unskilled personnel to operate the machine.

- ◆ Always use a high quality aluminium oxide (it is strongly recommended to use **MESTRA aluminium oxide**). A very low quality aluminium oxide can produce obstructions in the nozzle.
- ◆ Make sure that the aspiration unit can suck abrasive particles. We strongly recommend the MESTRA Eolo Plus R-080531 aspiration unit.
- ◆ Connect the power cord to a grounded 230 V, 50/60 Hz power source.
- ◆ In case of leaks in the pressured air circuit, consult the installation engineer immediately.

MAINTENANCE OF THE PISTOL FLEX

Do not bend or twist excessively the flex that holds the pistol and allows positioning it. It could break and then the abrasive would not go up the pipe. A spare flex is supplied with the unit. When needed, replace it following these steps:

1. Loosen the two lower bolts in the back cover and remove the cover.
2. Undo the cap nut and the normal nut that hold the flex from the back cover. They may be tight due to the sealant.

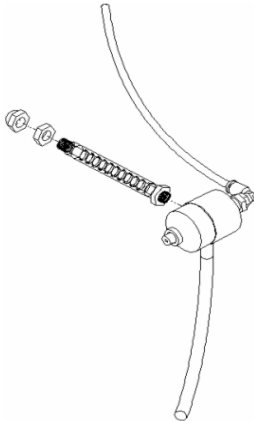3. Disengage the white pipe from the pistol by pulling it.

4. Disengage the blue pipe from the pistol by pulling it while you press the adaptor.
5. Unscrew the flex from the body of the pistol with a pair of pliers. It may be very tight due to the sealant. Hold the flex as near the pistol as possible.
6. Screw in the spare flex. Apply a sealant to prevent air leaks. Do not twist the flex. Hold the flex as near the pistol as possible.
7. Engage the two pipes again.
8. Fix the flex to the back cover with the two nuts. Apply a sealant to prevent air leaks.

UNIT CONTROLS PLASTIC PROTECTOR

In order to protect the unit controls from abrasive powder, you can stick the supplied plastic protector on the housing (see the picture).

TECHNICAL SPECIFICATIONS

Height:	505 mm
Width:	440 mm
Depth:	385 mm
Weight:	19 kg
Working pressure:	2 - 5 bar 0.2 - 0.5 MPa
Maximum air consumption:	90 litre/min at 5 bar
Voltage:	230 V, 50/60 Hz
Power consumption:	15 W

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