
## R-080539

ES .....	UNIDAD DE ASPIRACIÓN GEMINI .....	6
EN .....	GEMINI ASPIRATION UNIT .....	8
DE .....	GEMINI ABSAUGGERÄT .....	10
PO .....	UNIDADE DE ASPIRAÇÃO GEMINI .....	12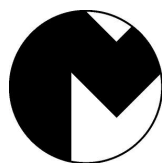

**MESTRA®**



TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

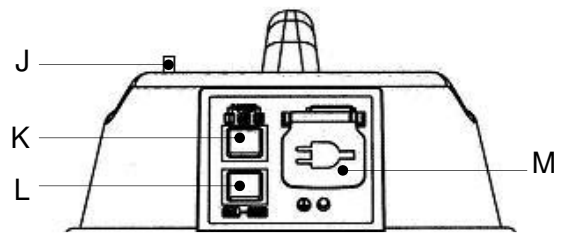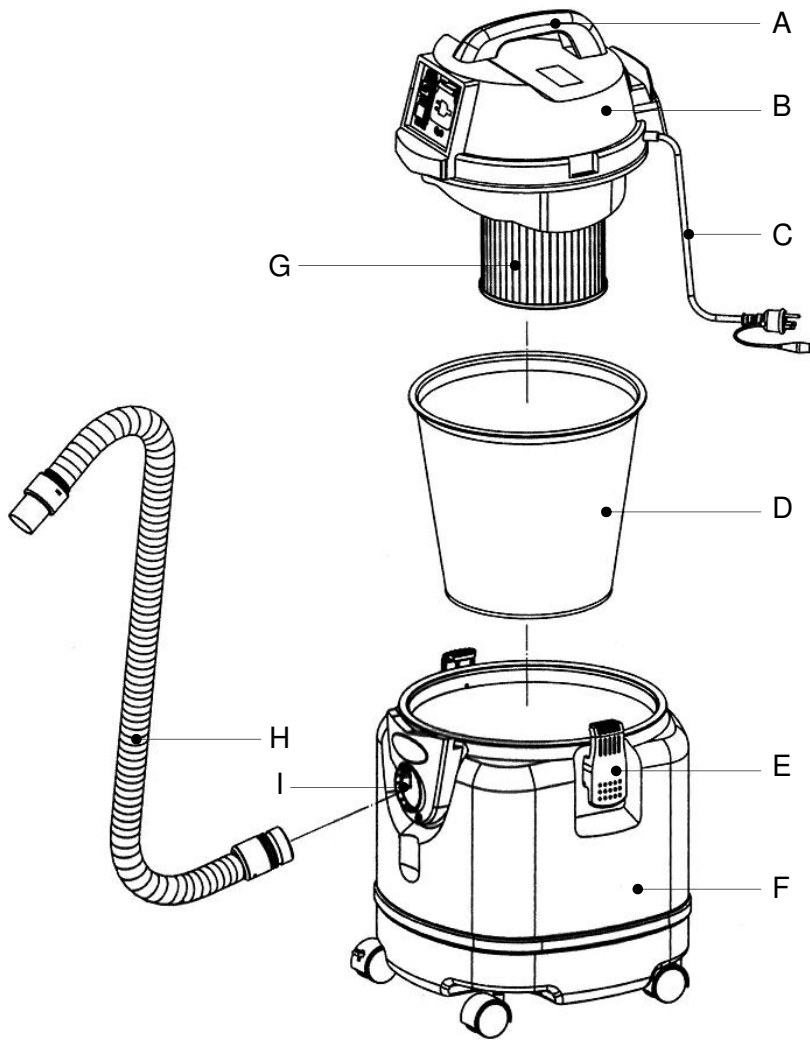A	Asa / <b>Handle</b> / <i>Asa</i> / Griff
B	Tapa / <b>Lid</b> / <i>Tampa</i> / Deckel
C	Cable de alimentación / <b>Power cord</b> / <i>Cabo de corrente</i> / Netzkabe
D	Bolsa filtro tela / <b>Filter bag</b> / <i>Filtro bolsa</i> / Gewebefiltersack
E	Cierre / <b>Lock</b> / <i>Fecho</i> / Verschluss
F	Tanque / <b>Tank</b> / <i>Tanque</i> / Staubbehälter
G	Filtro de papel / <b>Paper filter</b> / <i>Filtro de papel circular</i> / Papierfilter
H	Manguera / <b>Hose</b> / <i>Mangueira</i> / Schlauch
I	Boca de aspiración / <b>Aspiration nozzle</b> / <i>Boca de aspiração</i> / Absaugöffnung
J	Potenciómetro / <b>Potentiometer</b> / <i>Potenciómetro</i> / Potentiometer
K	Interruptor On-Off / <b>On-Off switch</b> / <i>Interruptor On-Off</i> / On/Off-Schalter
L	Interruptor Automático-Manual / <b>Auto-Man switch</b> / <i>Interruptor Automático-Manual</i> / Schalter Automatik-Manuell
M	Toma de corriente / <b>Power outlet</b> / <i>Toma corrente</i> / Stromanschluss

No usar con las manos húmedas.  
**Do not use with wet hand.**  
*Não use com as mãos molhadas.*  
Nicht mit feuchten Händen verwenden.

Limpiar el tanque después de cada uso, manteniéndolo seco para la próxima vez.  
**Clean the tank after each use and keep dry for next time.**  
*Limpe o tanque após cada utilização, mantendo-o seco para a próxima vez.*  
Staubbehälter nach jedem Gebrauch reinigen und für das nächste Mal trocken halten.

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## INSTALLATION

1. Carefully unpack your Gemini aspiration unit. You will find a flexible connecting hose inside the packing.
2. Choose a firm surface to place the unit, far from heat sources, damp or vibrations.
3. Open the lid (B) and make sure the filter-bag is correctly placed. Close the lid again.
4. Connect the unit's aspiration hose to the aspiration nozzle (I). Firmly turn it clockwise. The other end of the hose can be connected to a peg, sand blast, dry trimmer or any other similar device.
5. Connect the power cord to a grounded 230V, 50 Hz power source.
6. The unit is ready to be operated.

### IMPORTANT

In case of liquid aspirating, remove first the paper filter (G).

## OPERATION

1. Select the operational mode with the switch [TOOL/VACUUM].
  - If you select the mode *VACUUM*, aspiration motor will immediately start after pressing the switch [ON/OFF].
  - If you select the mode *TOOL*, the unit will work synchronously with a tool previously connected to the power outlet (M).


**NOTE:** Min. power need for the automatic detections is 100 W.  
Max. power supported by the power outlet (M) is 750 W.

2. Press the switch [ON/OFF]. The unit will start if you selected *VACUUM*. It will keep in stand-by if you selected *TOOL*.
3. You can regulate the motor power (from 650 to 1200 W) by turning the knob (J).

## UNIT EMPTYIGN - FILTER CLEANING

Periodically, check the tank filling level. Empty and clean it if necessary in the following way:

1. Disconnect the unit from the power supply.
2. Open the two lateral locks (E).
3. Pull the handle (A) upwards to remove the lid (B).
4. Remove the filter-bag (D).
5. Remove dust inside the tank (F).
6. Clean the round filter (G) or replace it with a new one and place again in its position. In order to remove the round filter, turn the knob (N) counterclockwise.

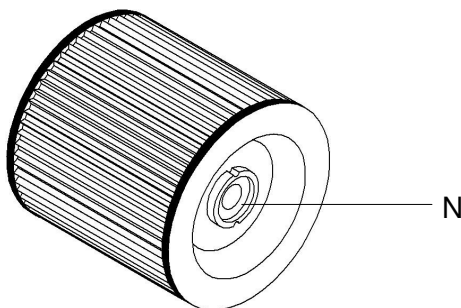7. Place the two filters in the original position again. Do not forget locking the round filter by turning the knob (N) counterclockwise.
8. Place the upper lid again and close the two locks (E).

## MAINTENANCE AND CLEANING

In order to achieve the machine's optimal performance and a long working life, we strongly recommend you:

- In case of liquid aspirating, remove first the paper filter (G).
- Clean the outer casing of the unit by always using a cloth. Do not use solvents or any other inflammable product.
- Periodically check the filling level of tank and filters.
- Replace the filters when damaged or if you can not clean them. Always use original spare parts.
- Make sure the aspiration hoses installed in your lab do not have losses due to cracks, breaks or faulty connections.

## PRECAUTIONS

- Before connecting the machine, make sure that the power source is a grounded 230 V, 50 Hz.
- Do not connect micromotors or other devices with a power consumption of more than 750 W in the Schuko power outlet.
- Do not allow children or unskilled personnel to operate the machine.
- Choose a rigid and well-balanced surface to place the aspiration unit.
- Do not open the upper lid when the unit is in operation.
- Disconnect the machine from the power source during the cleaning and filter replacement operations.

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# CE DECLARATION OF CONFORMITY

Gemini Universal Corp.

**HEREBY STATES:** That the **"GEMINI" ASPIRATION UNIT ref. 080539** with the following characteristics:

Power supply:	230 V, 50/60 Hz
Motor:	1200 W
Regulation:	650-1200 W
Detection:	>100 W
Noise:	68 dB
Capacity:	36 l
Height:	633 mm
Width:	406 mm
Depth:	433 mm
Weight:	10.4 kg

And with the following intended use:

**Aspiration of small particles at the dental laboratory.**

Meets the requirements set out in Directive 2006/95/EC concerning low voltage and in Directive 2004/108/EC concerning electromagnetic compatibility.

In witness whereof, the company issues this declaration of conformity.

Date: Aug. 20, 2012

Charge: Sales Manager

Signature:

A handwritten signature in black ink, appearing to be 'M. J. ...', written over a horizontal line.