TECHNOPAINT

Descripción

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TECHNOPAINT

Description

Fine investment material for partial denture technique

Uses

Fine investment for painting the wax or plastic patterns in the frame work technique. Application before filling the cylinder with the usual investment

Advantages:

- Avoids the defects on the castings
- It assures a fine casting surface
- Quick setting (1 minute)

Use instructions

Due to the composition of the product, during long storage periods, the product can settle. It's recommended to shake it well until a uniform product is obtained.

Apply the product with a brush onto the wax patterns.

Let dry completely approximately for 1 minute

Fill the cylinder with the usual phosphate or polysilicate investment.

Close the bottle after each use

If the product is too thick, some distilled water can be added until the correct viscosity is obtained.

Safety cautions

Work in well ventilated places

Do not eat, drink or smoke while using

Avoid contact with skin or eyes

In the event of ingestion, consult a doctor

For further information, see the safety data sheet of the product

Storage

The stability of the product is 18 months respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

Storage: Keep the product tightly closed in its original package and in a dry cool place.

Packing

6650-020 Bottle 200 ml

Additional information

The information given in this sheet is based on our level of knowledge at the moment at the date of printing and is not intended to be a warranty for the product.

protechno

Polígono Empordà Internacional
17469 VILAMALLA, GIRONA , SPAIN
Tel.: (34) 972 52 61 69
Fax: (34) 972 52 55 86
protechno@protechno.es
www.protechno.es