# ACRYSELF RX

---

RESINA ACRILICA RADIOPACA PER PALATI.

RADIOPAQUE ACRYLIC RESIN FOR PALATES.

RESINA ACRÍLICA RADIOPACA PARA PALADARES.

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## (GB) RADIOPAQUE ACRYLIC RESIN FOR PALATES

### DIRECTIONS:

Acrylic powder for the preparation of radiopaque dental prosthesis bases for diagnostics.  
To be used with Acry Self liquido.

### CONTRAINDICATIONS:

- Direct contact of non-polymerised material in the intraoral area.
- Direct use in the oral cavity.
- In case of proven allergy to components of Acry Self RX and Acry Self liquido.
- All uses not indicated.

**PREPARATION:** Shake the powder before use; the recommended mixing ratio is 20g of powder and 10ml of liquid (Acry Self liquido). It is also possible to pour the desired quantity of liquid (Acry Self liquido) into a suitable container, then saturate with the powder until a small quantity of liquid remains on the surface. Mix for approximately 20 seconds. The mixture can be used immediately, or you can wait for around 2 minutes for the mixture to become more plastic, and from this point it remains workable for around 5 minutes to be able to press it or apply it with a free hand trowel. To facilitate adhesion of the mixture, roughen and moisten the acrylic teeth and / or other acrylic parts with liquid Acry Self before application.

**RECOMMENDED POLYMERISATION:** Polymerisation at a temperature of 22°/23°C starts approximately 12-15 minutes after mixing. It is recommended to carry out polymerisation in a pot at a pressure of 2 ATM for 10 minutes at a temperature of 45°C. NOTE: The ambient temperature has a significant effect on the rest and plasticity of the dough, leaving the final result unchanged.

**FINISHING AND POLISHING:** Finish as usual, with burs, rubber points, brushes and with pumice.  
Mirror polish with the help of brushes and polishing products (Acry Lux Crema, Acry lux Liquido, Acry Lux Pasta).

POWDER: Store in a cool and dry place.

LIQUID (Acry Self liquido): Store at a maximum temperature of 30°C.

The product is designed for professional use in the dental sector. Follow the instructions for use. The manufacturer shall accept no responsibility for damages arising from different or inadequate use. The user must personally check the suitability of the product for the intended use, particularly if these uses are not reported in the instructions for use. Do not use after expiration. Warning: Any serious accident that may occur with the device must be reported by the user to the manufacturer and the competent authority of the Member State where the user and/or patient are established.

LIQUID (Acry Self liquido): Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. Store in a well-ventilated place. Keep cool. Dispose of contents/container in accordance with local/regional/national/international regulations.

Contains methyl methacrylate, ethylene dimethacrylate, N-N-dimethylparatoluidine.

POWDER: Do not breathe dust. IF ON SKIN: Wash with plenty of soap and water.  
Contains benzoyl peroxide, methyl methacrylate. May produce an allergic reaction.

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