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Instructions for use

**Gebrauchsanweisung****Mode d'emploi****Product description:**

Calcicur is a radiopaque, ready-to-use water-based calcium hydroxide paste with a pH value of >12.5.

Indications:

- indirect capping for the treatment of caries profunda
- direct capping when the pulp is opened or a pulpotomy is performed
- lining of cavities to prevent exposure to acid media when using cements
- for temporary filling of root canals

Dosage instructions and method of use:

Clean and dry the cavity before applying **Calcicur** paste in thin layers until the desired thickness is reached. Allow each layer to dry briefly or dry carefully with an air syringe. Do not apply paste to the cavity margins and remove any material adhering to them. Then place lining and filling.

For root canal fillings, first prepare and clean the root canal (e.g. hydrogen peroxide 3 % and sodium hypochlorite solution) and then insert paste into the root canal with the enclosed application cannula or with a Lentulo. Then move the paste towards the tip of the root using the Lentulo.

Take care to avoid trapping air or pushing the material beyond the apex, then place the filling. Checking the temporary root canal filling by x-ray is recommended. Renew the temporary root canal filling after extended retention time (more than 6 - 8 months).

Temporary filling of the root canal with **Calcicur** can be carried out either before or after complete root maturation.

Supplementary information/precautionary measures:

Improper storage can cause drying out or separating of the paste.

Calcium hydroxide is caustic to skin and mucous membranes. Avoid contact with the eyes!

In the event of accidental contact, rinse thoroughly with water and seek medical advice if necessary.

Those areas of the hard tooth substance which will be adhesively bonded to dental materials must not be treated with calcium hydroxide. Do not apply **Calcicur** in thick layers or as a permanent filling or luting material.

Storage:

After use, remove the application cannula from the syringe and re-seal with the protective cap.

Do not use hardened paste.

Store at temperatures of 15 °C to 23 °C.

Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1093 E1 0714 99 © by VOCO
VOCO GmbH