ventura etching gel

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EN INSTRUCTIONS FOR USE

1. DEFINITION

Ventura etching gel is a thixotropic acid gel for the acid etching technique for dental enamel. Its consistency and its presentation, in syringes with fine dispensing tips, permits easy and safe application, limited to the areas you wish to condition.

2. COMPOSITION

Ventura etching gel contains: 37% Orthophosphoric acid.

3. INDICATIONS

Conditioning of dental enamel with the acid etching technique to provide mechanical adherence in direct restoration processes with resin base materials, or in preventive sealing processes of pits and fissures.

4. CONTRAINDICATIONS

None known.

5. SIDE EFFECTS

None if applied according to the instructions for use.

6. INTERACTIONS

None known.

7. INSTRUCTIONS FOR USE

7.1. Preparation

Clean the dental pieces to be treated.

7.2. Special Precautions

The use of protective glasses is recommended for patients, dentists and assistants when using the syringe dispenser system.

7.3. Preparing the dispensing system

Remove the cover from the syringe and put it away. Securely screw a dispensing tip to the syringe. Keeping the tip at a distance from the patient and dental staff, dispense a small quantity of gel on a tray or gauze in order to check that the dispensing system is not obstructed.

If an obstruction exists, remove the dispensing tip and dispense a small quantity of gel directly from the syringe. Remove any visible obstruction from the tip of the syringe. Put again the dispensing tip and dispense the gel. If the obstruction still exists, throw away the dispensing tip and replace it with another.

7.4. Storage of the dispensing system

Remove the dispensing tip and throw it away. Place the cover on the syringe. Do not store the syringe with the used dispensing tip or without placing the cover since this may cause obstructions.

7.5. Alternative application methods

If preferred, the gel can be dispensed from the syringe onto a mixing tray and applied with a disposable fine tip applicator or a brush. If a liquid conditioner is required, the gel can be dispensed in a Dappen glass and shaken to increase its fluidity.

7.6. Application

Carefully apply **Ventura etching gel** to the surface of the cavity, starting at the edges of the enamel and then spreading to the dentin, when the adhesion technique indicates it. Let the conditioner gel act for at least 15 seconds on the enamel and never more than 15 seconds on the dentin. For the enamel of primary or very fluoridated teeth only, we recommend etching up to 90 or a maximum of 120 seconds. Next, wash thoroughly for 15 seconds under air extraction. It is essential for a good adhesion that the enamel not be contaminated. If this should occur, the etching, washing and drying must be repeated.

8. EXPIRY

When stored below 23 °C (73 °F) the shelf life of **Ventura etching gel** is up to three years.

9. MARKING OF THE PACKAGING

The lot number and expiry date are printed on each package.

10. PRECAUTIONS

Corrosive. Avoid the contact of **Ventura etching gel** with mucous membranes, skin and eyes. If applied to dentin, do not allow it to act for more than 15 seconds and rinse thoroughly. **Ventura etching gel** should never be exposed to temperatures higher than 25 °C (77 °F), direct solar radiation or other heat sources. Close the container of **Ventura etching gel** immediately after use. **Ventura etching gel** should not be used after the date of expiry.

KEEP OUT OF REACH OF CHILDREN

11. EMERGENCY PROCEDURES

Contact with the skin : wash with soap and water.
Contact with the oral mucous membrane : rinse with large quantities of water.
Contact with the eyes : wash immediately with large quantities of water and consult with an ophthalmologist.
Ingestion : Seek immediate medical attention. Do not induce vomiting. Drink large quantities of water or milk. Take an antacid.

12. HYGIENE

Use the application instruments for only one patient. Clean and disinfect all other instruments after each use.

13. ADDITIONAL NOTES

The material has been developed for exclusive use by dental professionals. Instructions for use issued in December 2017 (2017-12).

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