ACRYTRAY

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(GB) SELF CURE RESIN FOR INDIVIDUAL IMPRESSION TRAYS, PLATES FOR THE REGISTRATION OF OCCLUSION AND FOR TEETH SET UP.

DIRECTIONS:

Self curing acrylic resin liquid and powder for individual impression trays, functional trays in cases of edentulism, bases for mounting removable protheses, occlusal plates.

CONTRAINDICATIONS:

- In case of proven allergy to components of Acry Tray.
- Direct contact of non-polymerised material in the intraoral area.
- All uses not indicated.

PREPARATION: Powder/liquid proportions: 20g Acry Tray powder to 5.5 g Acry Tray liquid. Pour the desired quantity of liquid into a suitable container (preferably plastic) and add the powder in the indicated proportion. Stir for about 20 seconds until the mixture can be easily removed from the container and then mix by hand for about another 40 seconds. The resin is then ready to be adapted to the plaster model previously insulated with Acry Film (or other sealant). Acry Tray is workable for approximately 3 minutes and polymerisation occurs in around 6 minutes. The resin is easy applied to the model because it is very plastic. After polymerisation, the edges are perfectly adherent to the plaster of the model.

N.B. all times refer to an ambient temperature of 23°C.

Powder: Store in a cool and dry place.

Liquid: Store at a maximum temperature of 30°C.

The product is designed for professional use in the dental sector. Follow the instructions for use. The manufacturer shall accept no responsibility for damages arising from different or inadequate use. The user must personally check the suitability of the product for the intended use, particularly if these uses are not reported in the instructions for use. Do not use after expiration. Warning: Any serious accident that may occur with the device must be reported by the user to the manufacturer and the competent authority of the Member State where the user and/or patient are established.

LIQUID: Highly flammable liquid and vapour. Harmful if swallowed. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. Store in a well-ventilated place. Keep cool. Dispose of contents/container in accordance with local/regional/national/international regulations.

Contains methyl methacrylate, ethylene dimethacrylate.

POWDER: May cause an allergic skin reaction. Do not breathe dust. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention.

Contains: benzoyl peroxide, methyl methacrylate.

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Ruthinium® - Dental Manufacturing S.p.A.