ACRYSELF

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(GB) SELF CURING ACRYLIC RESIN FOR TOTAL AND PARTIAL REMOVABLE PROSTHESES, REPAIR AND RELINING. INDICATED FOR THE INJECTION TECHNIQUE.

DIRECTIONS:

Self curing acrylic resin liquid and powder with a high molecular weight for total or partial removable prostheses, combined prostheses, implant-supported prostheses, relinings, and for repairs of removable prostheses. It can also be used in the injection technique.

CONTRAINDICATIONS:

- Direct contact of non-polymerised material in the intraoral area.
- Direct use in the oral cavity.
- In case of proven allergy to components of Acry Self.
- All uses not indicated.

PREPARATION: the recommended mixing ratio is 20g of powder and 10ml of liquid.

It is also possible to pour the desired quantity of liquid into a suitable container, then saturate with the powder until a small quantity of liquid remains on the surface. Mix for approximately 20 seconds. The dripping mixture can be used immediately, pour the mixture into the syringe in case you want to inject it or you can wait for around 3 minutes for the mixture to become plastic, and from this point it remains workable for around 5 minutes. To facilitate adhesion of the mixture, moisten the edges of the repair with Acry Self liquid before application.

RECOMMENDED POLYMERISATION: Polymerisation at a temperature of 22°/23°C starts approximately 12-15 minutes after mixing. It is recommended to carry out polymerisation in a pot at a pressure of 2 ATM for 10 minutes at a temperature of 45°C.

FINISHING AND POLISHING: Finish as usual, with burs, rubber points, brushes and with pumice.

Mirror polish with the help of brushes and polishing products (Acry Lux Crema, Acry Lux Liquido, Acry Lux Pasta).

NOTE: The ambient temperature has a significant effect on the rest and plasticity of the dough, leaving the final result unchanged.

Powder: Store in a cool and dry place

Liquid: Store at a maximum temperature of 30°C

The product is designed for professional use in the dental sector. Follow the instructions for use. The manufacturer shall accept no responsibility for damages arising from different or inadequate use. The user must personally check the suitability of the product for the intended use, particularly if these uses are not reported in the instructions for use. Do not use after expiration. Warning: Any serious accident that may occur with the device must be reported by the user to the manufacturer and the competent authority of the Member State where the user and/or patient are established.

LIQUID: Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. Store in a well-ventilated place. Keep cool. Dispose of contents/container in accordance with local/regional/national/international regulations.

Contains methyl methacrylate, ethylene dimethacrylate, N-N-dimethylparatoluidine.

POWDER: Do not breathe dust. IF ON SKIN: Wash with plenty of soap and water.

Contains benzoyl peroxide, methyl methacrylate. May produce an allergic reaction.

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