idobase SOFT

ES

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E

UNIDESA introduces **IDOBASE SOFT**, heat cure acrylic resin for production of temporary soft relining for removable dentures by means of the hot curing technique..

IDOBASE acrylic resin has a special formulation to achieve the best results in an easy and practical way.

It is available in pink & pink veined.

IDOBASE acrylic resin is cadmium free.

Instructions:

Mixing ration: 10 gr. of powder – 9 ml of hot cure liquid.

- Mix at room temperature (21-23°C) and leave it for 30 seconds.

- Mix then again strongly and leave it for 10 minutes.

Handling time is subject to surrounding room temperatures, at 23°C IDOBASE SOFT is packable for 1/2 hour after mixing.

-Pack it in a flask, press it and put it in a clamp, immerse the flask in cold water.

- Bring it to boiling temperature and boil it for 30 minutes.

- Cool slowly.

Flasking: For flasking the waxed-up dentures use conventional detail compression moulding methods and normal hard plaster material.

Finishing and polishing: The denture should be finished and polished in the usual manner.

PRECAUTIONS:

This product is not resistant to chemical oxidant agents or strong acids.

During processing of cured material avoid breathing in dust and fumes.

The product is not visible by x-rays.

Do not reuse this product.

Product specifically designed for removable prosthesis

Do not modify mixing ratios or curing times or temperatures as this may affect the final result. Do not mix the product (powder / liquid) with other different products.

Do not contaminate during processing.

Acrylic resin: In case of an allergic reaction, stops further application and seek medical advice

HAZARD DATA SHEET

Powder: Store in a cool, dry place away from direct sunlight.

R 42/43: May cause sensitization by inhalation or skin contact

R 36/37: Irritant for eyes and respiratory vies

Liquid: Store below 30°C in a dry place, away from direct sunlight.

R11: Highly flammable.

R36/37/38: Irritant to skin, eyes and nose.

R43: May cause sensitization by skin contact

S9: Store in a well-ventilated place.

S16: Keep away from sources of ignition – No smoking

S26: In case of contact with eyes, wash immediately with water and seek medical assistance.

S29: Do not empty into drains

INFORMATION FOR THE PATIENT

The patient should be informed of the following points:

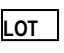
- Normal oral hygiene must be maintained.
- The material is not visible at x-ray.
- Material is resistant to abrasion but heavy bites should be avoided.
- In case of an allergic reaction, seek medical advice.

The material is for dental use only.

The manufacturer is not responsible for damage caused through improper use or any other use than those explains in these instructions.

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Explanation of symbols used on the label

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