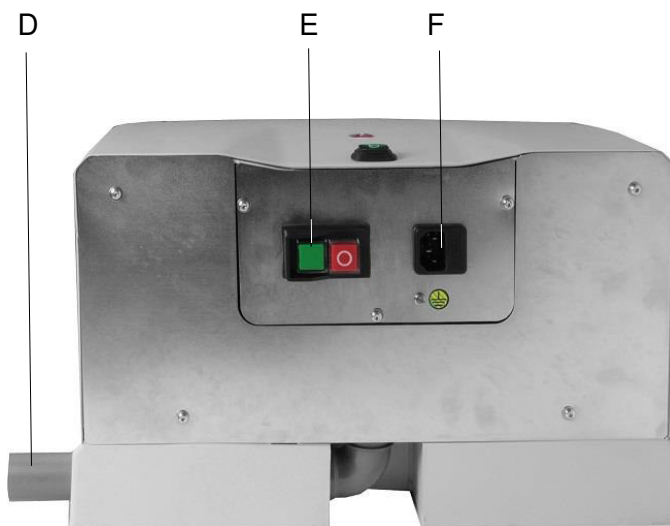
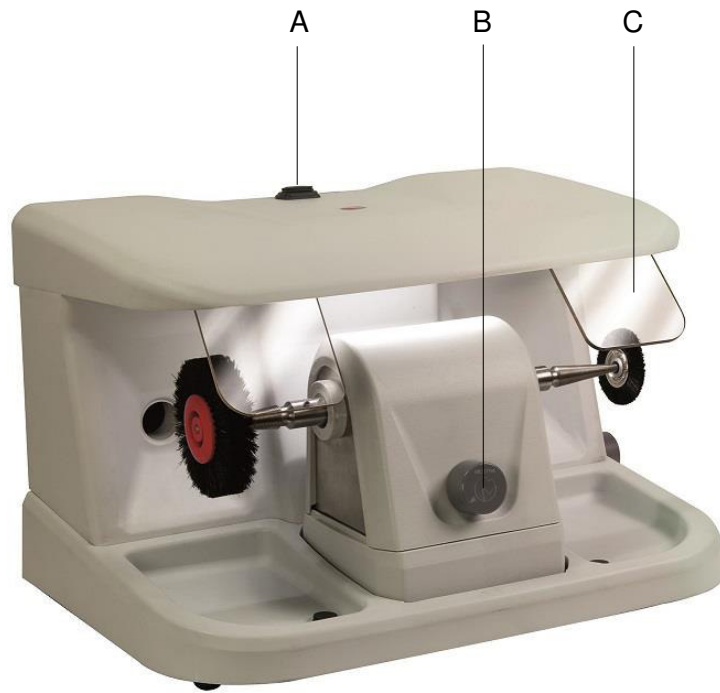
R-080508

GORBEA

ES.....	3	Manual original
EN.....	5	

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Ignacio Mestraitua
 Director Gerente
 Sondika, 14.03.2017

NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

The R-080508 is a simple polishing box especially designed to polish and grind small items in the field of dental mechanics.

Installation

1. Carefully unpack the unit. Check that it has not sustained damage during shipment.
2. Place the unit on a flat and firm surface remote from any source of heat or vibration.
3. Both brush holders are factory assembled and adjusted. They must not be disassembled.
4. Connect the unit to an adequate earthed power source according to the label.

Fitting the brush to the brush holder

1. Insert the brush into the brush-holder, ensuring that the brush-holder shaft is perpendicular to the plane of the brush.
2. Screw the brush onto the brush-holder as tightly as possible by hand.
3. Visually check that the brush-holder shaft is perpendicular to the plane of the brush by spinning the assembly in your hand.
4. Start up the unit: the spin of the motor will reveal whether the wheel or brush is off line. If there is more than 2 mm of slip, dismount and re-fit the wheel-holder or brush-holder.
5. Remember that the friction torque produced during work after a brush is fitted to the brush-holder will keep it tight. This effect can be favoured by a suitable workpiece attack angle.

Conditions of use

For best results use the unit as follows:

- Mount an appropriate brush-holder/wheel-holder as indicated above.
- Connect the unit to an adequate earthed power source according to the label.
- Press the green button (E) to connect the power.
- Press the switch (A) to turn the light on.
- Select the desired spin speed (2000-7000 rpm) turning switch (B).
- Before polishing or grinding a workpiece, check the motor spin direction, as the initial workpiece attack angle depends on this. Incorrect attack angles can be extremely dangerous.
- The following protective equipment is recommended for polishing or grinding:

- safety goggles;
 - safety gauntlets;
 - face-mask;
 - proper exhausters;
 - proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
 - Be careful with long hair.
 - No objects similar to ropes or chains that could be hooked by the machine.
- The unit has a protective system to prevent sudden start-ups (E). If the electrical power supply to the polisher motor is cut off it stops, but when power is restored the machine might start up unexpectedly and anyone manipulating the motor shafts at the time might suffer injury. To prevent this Talleres Mestraitua, S.L. has fitted the unit with an automatic switch which turns it off in case of a power failure. If this happens during operation the device will have to be reset before work can continue.
 - Clean the unit after using it.

Maintenance and cleaning

To ensure a long, trouble-free useful life for your unit, proceed as follows:

- To clean the motor and base use a cloth. Never use alcohol, solvents or other inflammable liquids.
- Never immerse the unit in any liquid.
- If you notice unusual noises or vibrations, send the unit to your technical assistance service.
- This unit is designed to work correctly on the four rubber legs.

Precautions

- Before use, check that the unit is connected to an adequate earthed power source according to the label
- Discard any cracked or damaged brush or grinding wheel: if these elements break during operation they could cause serious accidents.
- Before starting work, ensure that the brush-holder/wheel holder is firmly attached to the motor shaft.
- When handling workpieces, use safety goggles, gauntlets, a face mask and the safety shields.
- Do not allow children or unskilled personnel to handle the unit.
- If the motor shaft locks during normal operation, disconnect the unit from the mains before attempting to release it by turning it manually.
- Never polish, brush or grind materials which may give off particles which are toxic if inhaled.
- Before starting to polish or grind a workpiece, check the motor spin direction: the attack angle depends on this direction.
- Never use the unit close to solvents, alcohol or other inflammable materials. Take all possible precautions when grinding materials which may give off sparks.

- Antes de la utilización del aparato, asegúrese que éste se encuentra asentado sobre una base firme y sólida.
- Never use the unit without proper exhausters.
- Keep the working area clean and tidy in order to prevent accidents.
- Wear proper working cloth. Loose cloth, hanging or long hair could be caught by the machine.
- Make sure that you switch the power Off before plugging the unit, in order to avoy sudden start of the motor.
- Disconnect the unit from power supply before changing the wheel.

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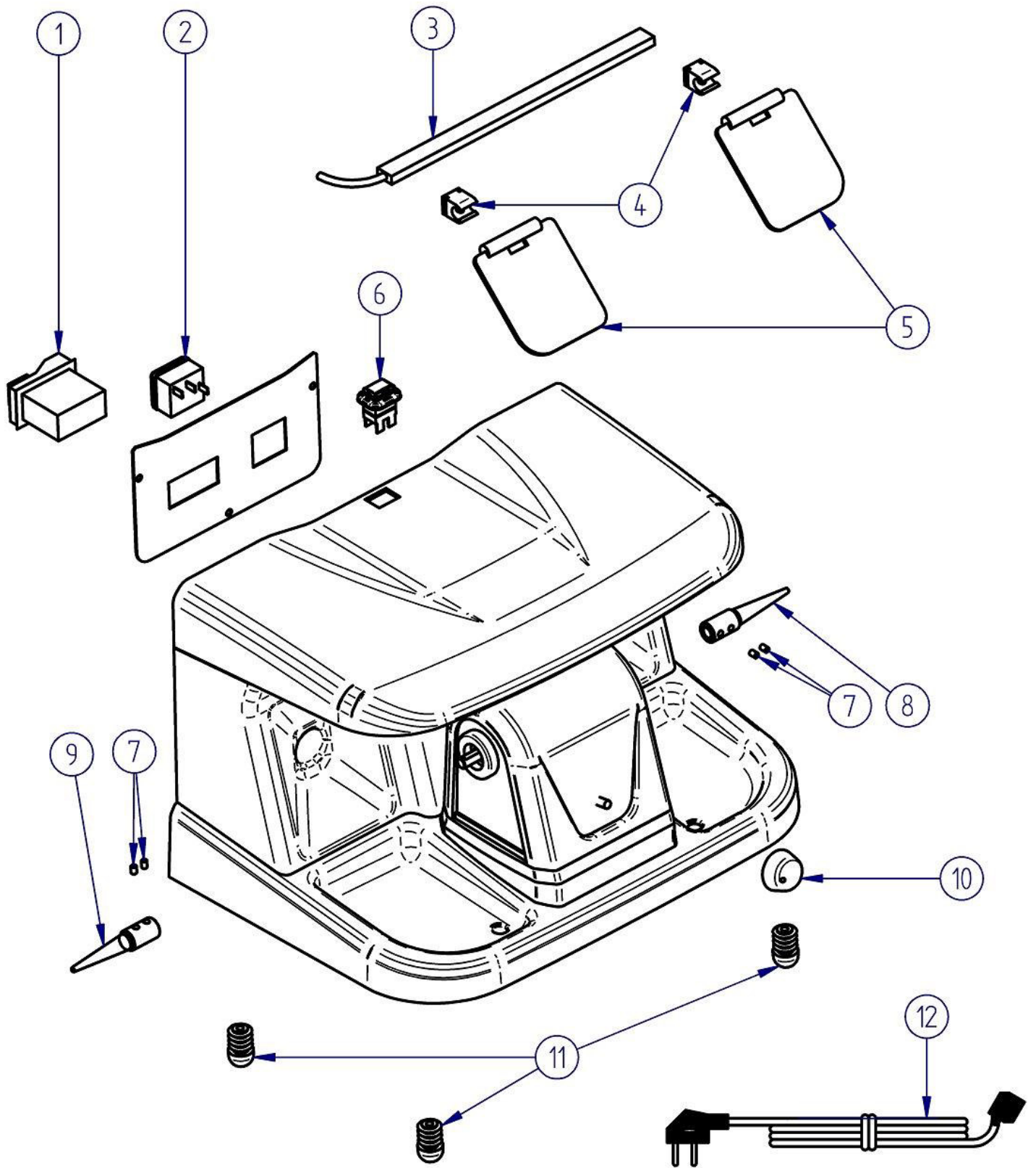
Meets the requirements set out in the following directives:

2014/35/UE (low tension)
 2014/30/UE (electro magnetic compatibility)
 2006/42/CE (machinery safety)

Ignacio Mestraitua
 Managing Director
 Sondika, 2017-03-14

Nº	Ref.	DESCRIPCIÓN	DESCRIPTION
1	080080-40	Interruptor de seguridad	Safe switch
2	080490-06	Base de conexión	Connection socket
3	080508-15	Barra LED	LED bar
4	080503-02	Pinza bisagra	Hinge clamp
5	080508-17	Pantalla	Shield
6	080422-17	Interruptor luz	Light switch
7	080164-11	Espárrago	Stud
8	100277-01	Porta cepillos derecho	Brush holder (right)
9	100277-02	Porta cepillos izquierdo	Brush holder (left)
10	080164-06	Pomo regulación	Regulation knob
11	080508-18	Pata	Leg
12	000250	Cable	Cable

MESTRA[®]

Your trusted brand

Talleres Mestraitua S.L.

Txori-erri Etorbidea, 60
T: (+34)944530388 - F: (+34)944711725
mestra@mestra.es - www.mestra.es
48150 SONDIKA - BILBAO - ESPAÑA