ACRYSELF P

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(GB) POURABLE SELF CURING ACRYLIC RESIN FOR TOTAL AND PARTIAL REMOVABLE PROSTHESES, REPAIR AND RELINING.

DIRECTIONS:

Self curing acrylic resin liquid and powder with a high molecular weight for total or partial removable prostheses, combined prostheses and skeletal prostheses with the casting technique. Also usable for repairs and relinings.

CONTRAINDICATIONS:

- Direct contact of non-polymerised material in the intraoral area.
- Direct use in the oral cavity.
- In case of proven allergy to components of Acry Self P.
- All uses not indicated.

PREPARATION: Mixing ratio: 2 parts of powder and one part of liquid (1.5:1 by weight). We recommend working at a temperature of 22 ° / 23 ° C. Mix the two components and stir for at least 20 seconds. From this point on, the castability time is approximately 3 minutes. During this time, the resin will be extremely fluid and can be poured into the "masks". The mixture will be shapeable with a spatula for approximately 5 minutes.

RECOMMENDED POLYMERISATION: It is recommended to carry out polymerisation in a pot at a pressure of 2 ATM for 10 minutes at a temperature of 45°C, while at room temperature polymerisation occurs in around 15 minutes.

FINISHING AND POLISHING: Finish as usual, with burs, rubber points, brushes and with pumice.

Mirror polish with the help of brushes and polishing products (Acry Lux Crema, Acry lux Liquido, Acry Lux Pasta).

N.B.: Observing the above procedures will result in a denture with high stability and excellent tensile strength.

Powder: Store in a cool and dry place

Liquid: Store at a maximum temperature of 30°C

The product is designed for professional use in the dental sector. Follow the instructions for use. The manufacturer shall accept no responsibility for damages arising from different or inadequate use. The user must personally check the suitability of the product for the intended use, particularly if these uses are not reported in the instructions for use. Do not use after expiration. Warning: Any serious accident that may occur with the device must be reported by the user to the manufacturer and the competent authority of the Member State where the user and/or patient are established.

LIQUID: Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing vapours. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. Store in a well-ventilated place. Keep cool. Dispose of contents/container in accordance with local/regional/national/international regulations.

Contains methyl methacrylate, ethylene dimethacrylate, N-N-dimethylparatoluidine.

POWDER: Do not breathe dust. IF ON SKIN: Wash with plenty of soap and water.

Contains benzoyl peroxide, methyl methacrylate. May produce an allergic reaction.

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Ruthinium® - Dental Manufacturing S.p.A.