GAPSEAL[®]

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Instruction for Use

Mode d'emploi

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Upute za uporabu

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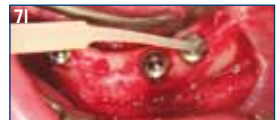
GAPSEAL®

A material for the sealing of empty spaces in implants to avoid periimplantitis

Application Area:

GapSeal is ideal for the sealing of capillary gaps and hollow spaces of all composed implants. It should be used from the very beginning, i.e. immediately after fitting of the implant when screwing in the locking screw so that the implant is already protected during the healing phase. It is recommended to use GapSeal again abundantly for each further opening of the implant, e.g. prior to screwing a healing cap, an impression pillar or an abutment. If possible, implants having an easy to remove superstructure should be filled during recalls and preventive check-ups.

Contraindications, Precautions and

Warning Notices:

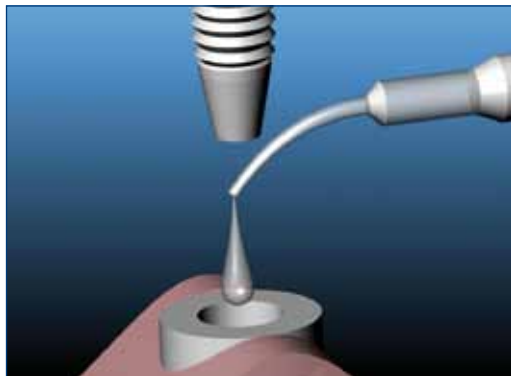
GapSeal should not be swallowed or get into contact with open wounds. In case of hypersensitivity against one of the ingredients, GapSeal should not be used.

If a cementing of supra-structures is intended, GapSeal has to be removed from the structure with alcohol or xylol before-hand, otherwise a proper cement adhesion is not guaranteed. The carpules are designed for onetime use. When reusing the carpule the functionality cannot be guaranteed and a high risk of infections occurs.

Do not sterilize GapSeal carpules before use.

Side Effects, Reciprocal Action with other Substances:

Side effects, reciprocal actions with medicaments or materials are not known and did not occur during the perennial clinical tests.

Composition:

Highly viscous silicone matrix with thymole.

Storage:

Store GapSeal dryly and protect from direct sunlight. Keep storage temperatures between 10 and 25 °C. If expiry date is exceeded or the sterile package is damaged, do not use GapSeal.

Delivery/Form:

GapSeal is delivered in sterile blister packs per 10 carpules each. The applicator is autoclavable up to 134 °C. For detailed information, see information on reprocessing.

Application:

Prior to clamping the 0.06 ml carpule into the applicator, the stamp is moved backwards by turning the transport wheel forward (pict. 1).

The insertion slot for the carpule is now released (pict. 2).

The cannula part is formed above the applicator shaft in such a way that it is slightly bent and not buckled (pict. 3).

Now the carpule is inserted into the insertion slot from the top (pict. 4).

After removal of the cap from the bent cannula (pict. 5), the stamp of the applicator is moved forward by turning the transport wheel backwards and GapSeal is released (pict. 6).

It is recommended to clean the implant thoroughly before insertion of GapsSeal (H_2O_2 , alcohol or xylene) and to dry it afterwards. The dry and clean inner space of the implants is then filled from the bottom up (pict. 7).

It is recommended to spread GapSeal abundantly also on the abutments and screws prior to insertion. The carpules are designed for a single use, mostly the content of one carpule is sufficient for 1–2 implants depending on the simultaneous use for supra-structures.

Characteristics:

GapSeal guarantees a sealing of the gaps and hollow spaces of implants thus avoiding auto- as well as re-infections by possible germ accumulation in the implant. It is free from antibiotics that are anyway of minor therapeutic value in such small quantities, but rather abet the resistance building. GapSeal remains durably viscous and can be removed (alcohol, xylene).

A negative effect on materials and tissue is not known. The indication is especially given for screwed supra-structures because of the gaps and threads, however, it is also recommendable for cemented works, as the cement is also attacked by bacteria and germs can penetrate the interior of the implants.

Furthermore, GapSeal facilitates the handling of small screws (locking or set screws for the screwing of supra-structures) as small screws can be fixed to the screwdriver, placed and screwed in safely due to the highly viscous material (pict. 8).

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GHIMAS S.p.A
HAGER & WERKEN GmbH & Co. KG