

3DP Type	Resin Type	Layer Height(mm)	Bottom Layer Count	Exposure Time/s	Bottom Exposure Time/s	Transition Layer Count	Waiting Mode During Printing	Rest Time Before Lift/s	Rest Time After Retract/s	Bottom Lift Distance/mm	Lifting Distance/mm	Bottom Retract Distance/mm	Retract Distance/mm	Bottom Lift Speed-mm/min	mm/min Lifting Speed	mm/min Bottom Retract Speed	mm/min Retract Speed
Anycubic Photon Mono 4 Ultra	Implant Model Grey	0,05	4	4,2	35	8	Resting Time	0	0,5	4+4	4+3	6+2	5+2	60+90	90+150	120+60	180+90
	Implant Model Yellow	0,05	4	3,2	30	8		0	0,5	4+4	4+3	6+2	5+2	60+90	90+150	120+60	180+90
Photon Mono Photon Mono 4K	Implant Model Grey	0,05	4	4,2	35	8	Resting Time	0	0,5	8	7	/	/	60	120	90	150
	Implant Model Yellow	0,05	4	3,2	30	8		0	0,5	8	7	/	/	60	120	90	150
Anycubic Photon Mono SQ Anycubic Photon Mono SE	Implant Model Grey	0,05	4	4,7	39	8	Resting Time	0	0,5	8	7	/	/	60	120	90	150
	Implant Model Yellow	0,05	4	3,6	34	8		0	0,5	8	7	/	/	60	120	90	150
Anycubic Photon Mono X2	Implant Model Grey	0,05	4	3,4	28	8	Resting Time	0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
	Implant Model Yellow	0,05	4	2,6	24	8		0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
Anycubic Photon Mono X 6K	Implant Model Grey	0,05	4	3,2	26	8	Resting Time	0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
	Implant Model Yellow	0,05	4	2,4	23	8		0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
Anycubic Photon M3 Plus Anycubic Photon Mono M5s	Implant Model Grey	0,05	4	4,3	36	8	Resting Time	0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
	Implant Model Yellow	0,05	4	3,3	31	8		0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
Anycubic Photon M3 Premium	Implant Model Grey	0,05	4	3,6	30	8	Resting Time	0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
	Implant Model Yellow	0,05	4	2,7	26	8		0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
Anycubic Photon Mono 4	Implant Model Grey	0,05	4	6,0	50	8	Resting Time	0	0,5	8	7	/	/	60	120	90	150
	Implant Model Yellow	0,05	4	4,6	43	8		0	0,5	8	7	/	/	60	120	90	150
Anycubic Photon Mono M5s Pro	Implant Model Grey	0,05	4	5,0	42	8	Resting Time	0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
	Implant Model Yellow	0,05	4	3,8	35	8		0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
Anycubic Photon Mono M7	Implant Model Grey	0,05	4	3,8	32	8	Resting Time	0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
	Implant Model Yellow	0,05	4	2,9	27	8		0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
Anycubic Photon Mono M7 Pro	Implant Model Grey	0,05	4	2,7	23	8	Resting Time	0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
	Implant Model Yellow	0,05	4	2,1	20	8		0	1	4+4	4+4	6+2	6+2	60+90	60+120	120+60	180+90
Anycubic Photon Ultra (DLP)	Implant Model Grey	0,05	4	5,0	42	8	Resting Time	0	0,5	8	7	/	/	60	120	90	150
	Implant Model Yellow	0,05	4	3,8	36	8		0	0,5	8	7	/	/	60	120	90	150
Anycubic Photon D2 (DLP)	Implant Model Grey	0,05	4	7,6	63	8	Resting Time	0	0,5	4+4	4+3	6+2	5+2	60+90	90+150	120+60	180+90
	Implant Model Yellow	0,05	4	5,8	54	8		0	0,5	4+4	4+3	6+2	5+2	60+90	90+150	120+60	180+90
Anycubic Photon P1	Implant Model Grey	0,05	4	2,9	25	8	Resting Time	0	1	4+4	4+4	6+2	6+2	90+120	120+180	150+60	180+90
	Implant Model Yellow	0,05	4	2,2	25	8		0	1	4+4	4+4	6+2	6+2	90+120	120+180	150+60	180+90